

 3DHISTECH	Pannoramic Viewer
	Release Notes

Participants:	Gerely László
Version:	1.15
Date:	2011-06-07

New features:

1. Does not accept 1.11 or older license versions:

1.11 or older application versions cannot be run under Pannoramic Viewer.

2. 64-bit compatibility

Pannoramic Viewer runs on various editions of Microsoft Windows 7 operation systems (32/64 bit).

3. Displays Pannoramic slides scanned with Pannoramic P250 FLASH slide scanner

4. Supports and handles Z-stack MRXS slides

5. ERGODRIVE 3 implementation

ERGODRIVE 3 is an ergonomic operating device for handling digital slides. Users can pan and zoom similarly to a regular microscope.

6. MRXS thumbnails can be seen in Windows Internet Explorer

7. Handles Floating License Dongle

Floating license can be used for running applications within a LAN (Local Area Network) segment. The dongle key must be plugged into a computer that can be accessed any time (switched on and continuously operating) and on which a free Panoramic Viewer has been installed.

8. Opens a Case stored in CaseCenter

Panoramic Viewer displays all slides in a case by one click from CaseCenter

9. Supports UTF-8 characters in slide names

Slide names may contain accented letters